
The Most Economical NOBLE Alloys are Now Available!

NOBLECROWN

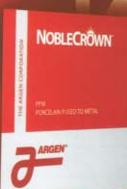





Pd 25.0% Co 42.6 20.0% Mo 12.0%

NOBLECRÔWN NF



Revolutionary New PFM Alloys WITH 25% PALLADIUM



Silver-Free

ADA Classification: NOBLE

Compatible with all major porcelains

Low Density: 9.0 g/cm3 NobleCrown 9.2 g/cm3 NobleCrown NF



PATENT PENDING









CANADA

Western Provinces

Quebec Province Ordent (514) 984-1808

Richard Nixon (866) 722-7436

Eastern and Ontario Provinces

Meric Dental Inc. (800) 373-3368

ARGEN

Alloy Makers to the World


THE ARGEN CORPORATION 5855 Oberlin Drive 5an Diego, CA 92121

Customer Service

(800) 255-5524

Technical Service (800) 255-5095

www.argen.com

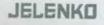

NOBLECROWN™ & NOBLECROWN™ NF Porcelain Alloy

NobleCrown™ & NobleCrown™ NF are white color, porcelain alloys formulated to be compatible with most dental porcelains. Superior physical properties facilitate the fabrication of single units, short and long span bridges.

COMPOSITION

Palladium	Nickel	Chromium	Molybdenum	Silicon
25.0%	37.5%	25.0%	12.0%	X
NOBLECROWN™ NF		35		
Palladium	Cobalt	Chromium	Molybdenum	Boron
25.0%	42.6%	20.0%	12.0%	Y

"X" denotes a content of less than one p

PHYSICAL PROPERTIES

Alloy	Melting Range	Casting Temperature	Density 9 g/cm ³	
NobleCrown™	2190-2300°F (1200-1260°C)	2550°F (1400°C)		
NobleCrown" NF	2280-2350°F (1250-1290°C)	2550°F (1400°C)	9.2 g/cm ³	

MECHANICAL PROPERTIES*

Alloy	Vickers Hardness	Yield Strength (0.2% Offset)	Ultimate Tensile Strength	Modulus of Elasticity	Elongation %	Coefficient of Thermal Expansion µm/m-K	
NobleCrown™	210	371 MPa	551 MPa	150 GPa	25%	25-500°C 14.7	25-600°C 14.9
NobleCrown™ NF	335	620 MPa	690 MPa	192 GPa	4%	25-500°C 14.4	25-600°C 14.8

* Properties displayed are after firi

INSTRUCTIONS FOR USE

Use conventional wax technique with minimal thickness of 0.3 mm. For single units use direct sprues with reservoirs. WAXING Bridges should use indirect spruing.

Use a high heat (phosphate bonded) investment. Follow manufacturer's instructions. The thickness of investment from the top INVESTING of the wax pattern to the top of the investment should be no more than 1/4" (6 mm).

End temperature is 1600°F (870°C). Maintain end temperature for 1 hour, plus 10 minutes for each additional ring. If using BURNOUT rapid fire investment, follow manufacturer's instructions.

CASTING Preheat a ceramic crucible in the furnace 10-15 minutes before casting. Use either a natural gas/oxygen or propane/oxygen torch with a multi-orifice tip. Regulate the mixture to use twice as much oxygen as fuel. Adjust the flame at the torch controls to get sharp bright blue inner cones approximately 1/8-1/4" (3-6 mm) long. Combustion will result in considerable sound. When melting several ingots, stack in piles 2-4 ingots high. Bring the flame to within about 1"-2" (3-6 cm) from the tip of the torch...do not touch the alloy with the bright blue inner cones. Begin heating the alloy, continuously moving the flame over each stack of ingots. The individual ingots will have a stiff, heavy oxide shell as they heat. When the stack slumps down on itself and there is essentially no further change to in the shape, it is ready to cast. Do not over heat to try to get the ingots to clear or melt together. Normal casting temperature for induction or electrical casting is 2550°F (1400°C). Some machines may need as much as 270°F (150°C) higher temperature setting. Use a ceramic crucible. Use at least 35% new alloy when remelting buttons.

FINISHING Grind the metal surfaces for porcelain application with non-contaminating aluminum oxide stones or cross-cut carbide bars. Blast with non-recycled 50 micron aluminum oxide. Clean in distilled water in an ultrasonic cleaner for ten minutes.

DEGASSING Place the units(s) in a preheated furnace at 1200°F (649°C) and raise the temperature to 1800°F (980°C) at 100°F (55°C) per min., vacuum. No hold time. Remove oxide by blasting.

For a better bond, fire a thin wash in vacuum at 15-20°F (10°C) above the regular opaque temperature, followed by a regular OPAQUING opaque coat.

SOLDERING NobleCrown: NP Presolder or LWNPNI for laser welding NobleCrown NF: Co/Cr Presolder or LWNPCO for laser welding